



Titanium Dental Technologies
YOU ARE OUR PRIORITY

Surgispon Dental

Absorbable Haemostatic Gelatin Sponge



Fast and effective



Absorbs 40 - 50 times of its own weight



Saves valuable surgical and anaesthetic time

Contact us



011 425 9819



accounts@ti-tec.com

Surgispon



SURGISPON® is a sterile hemostatic absorbable gelatin sponge made of pure pharmaceutical gelatine (lyophilized hydrolyzed collagen) foam, which gives it a uniform porosity & reacts neutrally. The dried foam is cut, packaged and sterilised using gamma radiation. **SURGISPON** is immediately available for use in the operating theatre and does not require any sterilisation by dry heat or autoclaving.

How to use Surgispon Dental

- Soak the cube in sterile saline solution or use dry
- If used dry, the sponge is slightly compressed. The sponges must be applied to a bleeding area under light pressure for one or two minutes until the bleeding stops
- If used wet, withdraw SURGISPON cubes and squeeze thoroughly between gloved fingers to expel air bubbles, replace in saline and keep there until needed
- When bleeding is controlled, the cubes can be left in situ

SURGISPON may be used in combination with antibiotics, Thrombin, chemotherapeutics without reduction of the sponges haemostatic effect. Carefully read applicable instructions for use before usages.

Contact us



accounts@ti-tec.com



011 425 9819

Products






Where to use Surgispon

To achieve immediate hemostasis in: -
Extractions & Oral bone and soft tissue surgery
(No Iodine & packed individually)




DETAIL
Dental cube place in cavity.

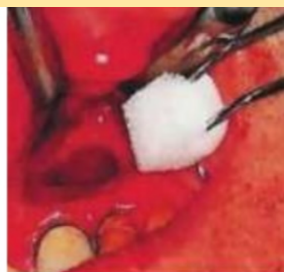

ENUCLATION OF CYSTS
Bleeding from the cavity is controlled with the dental cube.



Cube

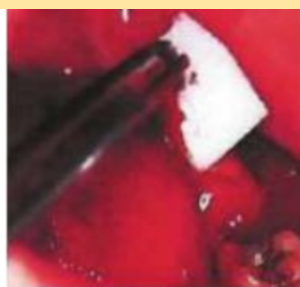

CONTROL OF PROFUSE
Bleeding after tooth extraction by means of several dental cubes



TOOTH EXTRACTIONS
Oozing bleeding from alveolus is stopped by application of SURGISPON® Dental cube.



Sheet



Tampon

Contact us



accounts@ti-tec.com



011 425 9819

Ordering Information

Product ID	Product	Dimensions (mm)	No. of Pieces per box
SSP-101010	CUBES	10x10x10	12
SSP-101010	CUBES	10X10X10	32
SSP-1208	DENTAL TAMPON	12X08 Ø	32
SSP-202007	SHEET	20x20x07	20